
Auto Suture™

Premium Extractor

1302756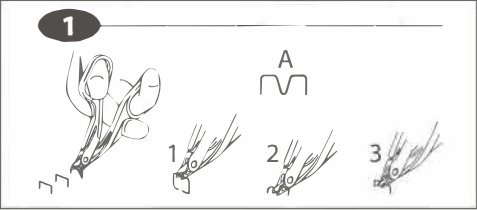

EN

BEFORE USING PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.

IMPORTANT!

This booklet is designed to assist in using this product. It is not a reference to surgical techniques. This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or resterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or resterilize this device.

WARNINGS AND PRECAUTIONS

1. This device is provided STERILE and is intended for use in a SINGLE procedure only. DISCARD AFTER USE. DO NOT RESTERILIZE.

1 INSTRUCTIONS FOR USE

NOTE: Do not pull up on staple before handle is fully squeezed.

1. Slip lower jaws of extractor under the backspan of the staple to be removed.
2. Close the instrument handles. This will unbend the staple.
3. Lift the extracted staple away from the skin and discard. Proceed to remove the next staple from the skin.

A) EXTRACTED STAPLE

AVOID PROLONGED EXPOSURE TO ELEVATED TEMPERATURES.

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